



EN

Epic White
2Go™

INSTRUCTIONS FOR USE

Whitening Gel (35% Carbamide Peroxide)

BY **BIOLASE**®

Epic White

2Go™

LIGHT ACTIVATED PROFESSIONAL WHITENING SYSTEM

INGREDIENTS

Glycerin, Urea Carbamide Peroxide 35% (11.7% released hydrogen peroxide), Propylene Glycol, Water, Carbomer, Potassium Nitrate, Triethanolamine, Sodium Saccharin, Potassium Sorbate, Flavor (aroma)

CONTENTS

- 1 Epic White 2Go LED Light
 - 2 Batteries
 - 1 Attachable Whitening Tray
 - 4 Epic White 2Go Professional Whitening Gel Syringes 3.5ml each (NET 14ml)
- (This kit provides 8-12 dual arch treatments.)



INSTRUCTIONS

Push open the bottom battery cover of the LED Light and remove the plastic covering from the battery.

To connect the Whitening Tray with the LED Light, insert one edge of the LED Light into the Whitening Tray and pull the flexible Whitening Tray over the other edge. The Whitening Tray should be flush and secure with the LED Light.

Dispense a thin layer of Whitening Gel along both front edges of the Tray – one thin layer for both upper and lower arches. Do not overfill the Tray to prevent soft tissue (gums) sensitivity.

Insert the Tray fully into the mouth, gently press the front edges with Whitening Gel onto the teeth, gently bite down on the soft Tray. Press the LED Light power button once to turn on the Light and to start the 10 minute timer. The power button

may be pressed again during the 10 minute cycle to turn the Light off. The LED Light will also gently vibrate to help spread the Whitening Gel and to provide a tactile sense of the treatment cycle.

NOTE: The recommended treatment time is 20 minutes. After the 10 minute timer turns the LED Light off, the light may be turned on again to complete the 20 minute treatment.

NOTE: Before removing the LED Light from the mouth, make sure the LED Light is off.

After treatment, remove the Whitening Tray, and rinse and brush the teeth with warm water to remove all remaining Gel.

Use only once per day. If sensitivity is felt, refrain from use for one day. If sensitivity persists, consult your dentist.

CAUTION

For professional use only. Not recommended if patient is pregnant, nursing or is under 18 years of age. Do not use if patient presents allergies to any of the ingredients. Avoid contact with eyes. Rinse immediately if product comes into contact with them. Store in a cool place. Refrigeration is recommended.

For each cycle of use, the first use is to be carried out only by the dental practitioner or under their direct supervision if an equivalent level of safety is ensured. Afterwards, the product can be provided to the patient to complete the cycle of use.

Reorder Whitening Gel System: PN 7450004

Order Whitening Gel Refill: PN 7450005

Manufactured by JD Oral Care MFG for BIOLASE

BIOLASE[®]

27042 Towne Center, Suite 270
Foothill Ranch, CA 92610 USA
+1 949.361.1200 | biolase.com

PN 5400106 Rev A Made in Canada