

Delight Your Patients, Faster.





5400578 Rev A






SYSTEM PARTS LIST epic


The laser system includes the following:

- 1. Laser Console (lithium ion battery pack already installed)
- 2. Screen Protectors (Peel-off clear screen cover qty. 30)
- 3. Delivery System (installed)
- 4. Assorted Surgical Tips
- 5. Surgical Handpieces (2)
- 6. Three (3) pairs of protective laser eyewear
- 7. DC power supply and power cord (one(1) US and one (1) International)

- Welcome Kit (Welcome Letter, User Manual, BIOLASE store information, Quick Setup Guide, Guide to Online Training & Product Registration Card, Limited Warranty Information)
- 9. Laser Warning Sign
- 10. Tip Initiation Kit
- 11. Remote Interlock cable
- 12. Philips-head screwdriver (for installing Footswitch batteries)
- 13. Footswitch
- 14. AAA batteries (2)


CAUTION: Federal Law restricts this device to sale by or on the order of a dentist or physician or other licensed medical practitioner.

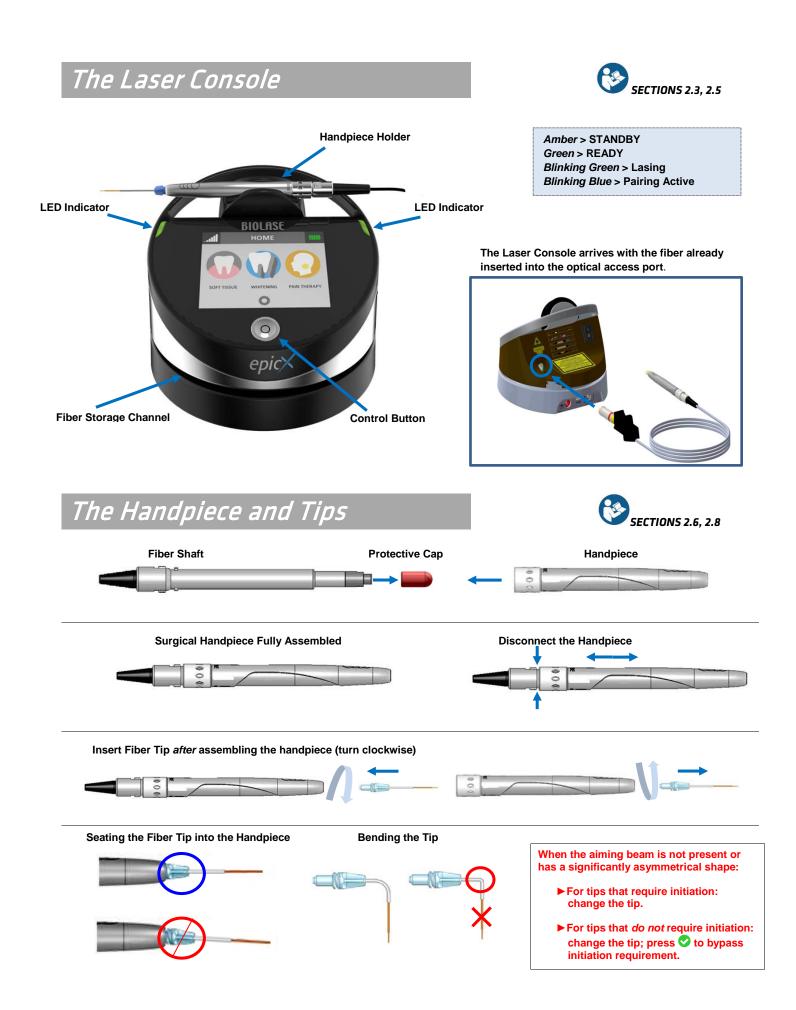


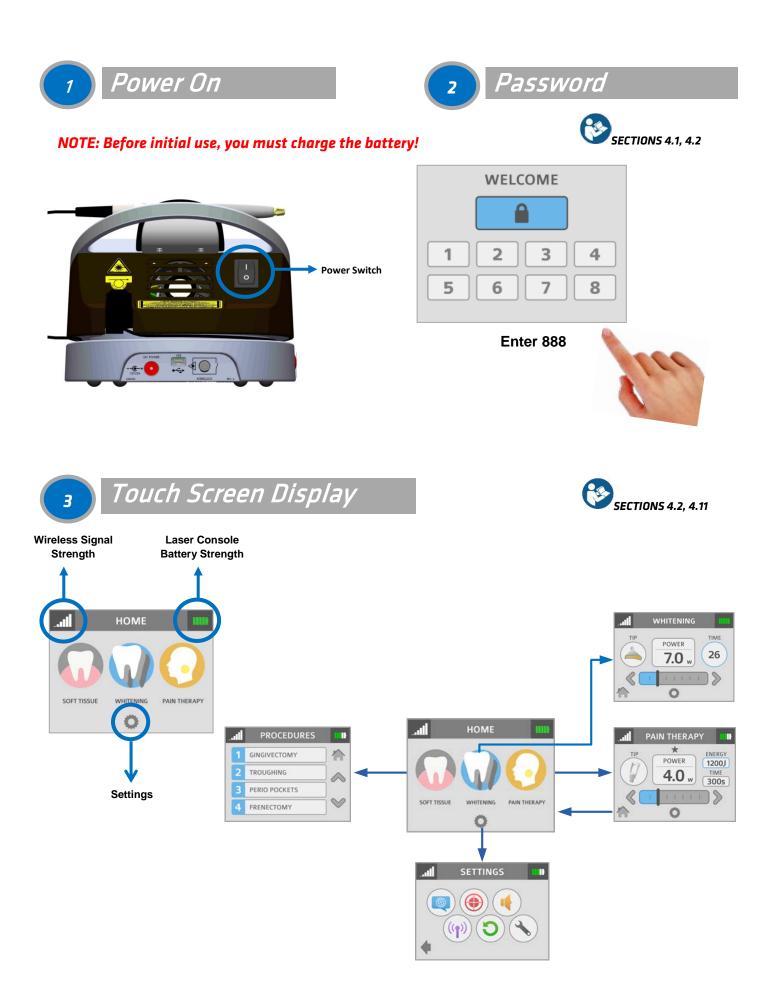





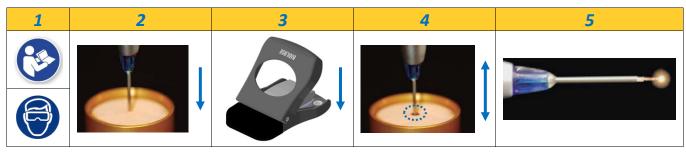

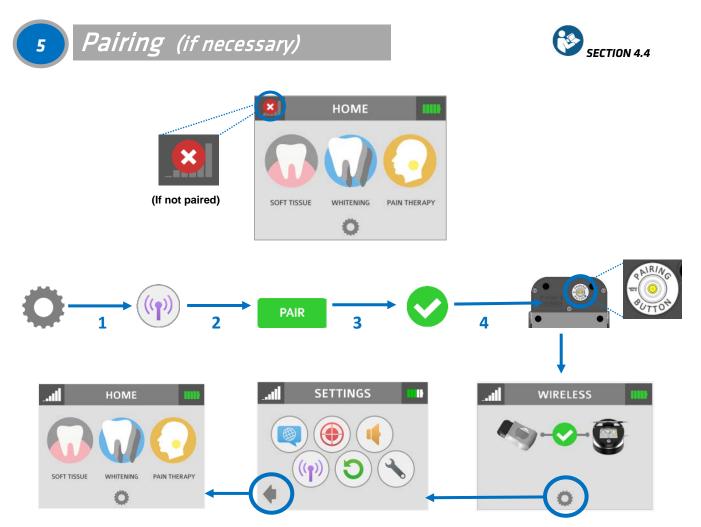

E	TION 11	Message !	Reason ?	Fix 🔽	
Error 1	ERICOU	THERMISTOR OPEN	THERMISTOR OPEN	CALL BIOLASE SERVICE	
Error 2	ERICON	THERMISTOR SHORTED	THERMISTOR SHORTED		
Error 3		SHUTDOWN TEMPERATURE	SYSTEM TOO HOT	ALLOW 5-10 MINUTES FOR LASER TO COOL DOWN	
Error 4		LASER CURRENT HIGH/LOW	OUTPUT IS OUT OF SPECS	CALL BIOLASE SERVICE	
Error 5	ERCO	FOOTSWITCH SHORTED IN STANDBY	FOOTSWITCH IS PARTIALLY PRESSED OR IS DAMAGED	PRESS/RELEASE FOOTSWITCH OR CALL BIOLASE SERVICE	
Error 6	ERRORI 7	ON/OFF BUTTON STUCK	КЕҮ STUCK	PRESS FRONT KEY	
Error 7	ERICON 7	FLASH CORRUPTED	MEMORY CORRUPTED	CALL BIOLASE SERVICE	
Error 8	ERRCHU	NO FIBER	FIBER NOT INSERTED	PLUG IN TRUNK FIBER	
Error 9	ERCAI	LOST FOOTSWITCH COMMUNICATION	WIRELESS INTERFERENCE	REPOSITION CONSOLE OR FOOTSWITCH TO IMPROVE COMMUNICATION	
Error 10	ERKON	EMERGENCY SWITCH	E-SWITCH PRESSED	PRESS E-SWITCH AGAIN	
Error 11	ERRCHU	REMOTE INTERLOCK	REMOTE INTERLOCK OPEN	CHECK REMOTE INTERLOCK CLOSED	
Error 12	ERR.CH)	BATTERY CRITICALLY LOW	BATTERY CRITICALLY LOW	PLUG IN DC SUPPLY	
Error 13	ERICON	INTERNAL ERROR	INTERNAL ERROR OCCURRED	RESTART UNIT	
Error 14	ERCOL	FOOTSWITCH BATTERY	FOOTSWITCH BATTERY CRITICALLY LOW	REPLACE FOOTSWITCH BATTERY	
WARNING 1		темр Нідн	SYSTEM IS HOT	ALLOW 5-10 MINUTES FOR LASER TO COOL DOWN	
WARNING 2		BATTERY LOW	BATTERY IS LOW	PLUG IN DC SUPPLY	
WARNING 3		BATTERY NOT CONNECTED	BATTERY NOT CONNECTED	PLUG IN BATTERY	
WARNING 4		FOOTSWITCH BATTERY LOW	BATTERY ON THE FOOTSWITCH IS LOW	REPLACE FOOTSWITCH BATTERY	
WARNING 5		FOOTSWITCH	FOOTSWITCH HELD	RELEASE FOOTSWITCH	
ALERT 1		WIRELESS NOT PAIRED	NO WIRELESS CONNECT	RE-ESTABLISH PAIRING (SEE SEC 4)	
ALERT 2		SYSTEM MUST BE IN READY MODE TO LASE	SYSTEM IS NOT IN READY MODE	PRESS THE CONTROL BUTTON IN ANY PROCEDURE SCREEN	



BIOLASE, Inc. 4 Cromwell Irvine, CA 92618 USA 949.361.1200 888.424.6527 biolase.com

ECREPMT Promedt Consulting GmbHAltenhofstrasse 80D-66386 St. Ingbert/Germany+49 6894 581020mt-procons.com



